




**AESKU.**DIAGNOSTICS  
THE DIAGNOSTIC TOOL THAT WORKS

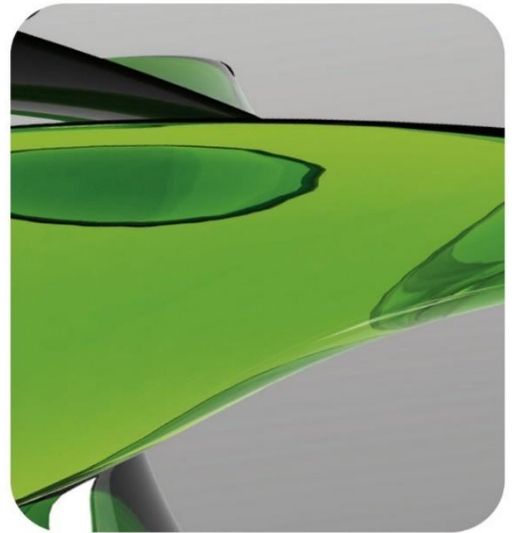

**AESKULISA<sup>®</sup>**  
THE DIAGNOSTIC TOOL THAT WORKS

# INSTRUCTION MANUAL

AESKULISA<sup>®</sup> Jo-1

REF 7113US





# Instruction manual

## Table of Contents

---

1	Intended Use.....	1
2	Clinical Application and Principle of the Assay.....	1
3	Kit Contents.....	2
4	Storage and Shelf Life.....	2
6	Sample Collection, Handling and Storage.....	3
7	Assay Procedure.....	4
8	Quantitative and Qualitative Interpretation.....	5
9	Technical Data.....	6
10	Performance Data.....	6
11	Literature.....	7
12	Regulatory Symbols.....	8
	Annex.....	9



## 1 Intended Use

**AESKULISA® Jo-1** is a solid phase enzyme immunoassay with recombinant human histidyl-tRNA-synthetase (HRS) for the semi-quantitative and qualitative detection of antibodies against Jo-1 in human serum. The assay is an aid in the diagnosis of polymyositis and dermatomyositis and should be used in conjunction with other serological tests and clinical findings.

## 2 Clinical Application and Principle of the Assay

Antibodies against Jo-1 are directed against the reactive site of histidyl-tRNA-synthetase (HRS) which is an cytoplasmic enzyme belonging to the group of aminoacyl transferases. These are responsible for the linking of the respective amino acid (for HRS it is histidine) to its cognate transfer RNA. HRS is present in the cell as homodimer, its identical subunits of approximately 50 kDa are each bound to tRNA.

Autoantibodies are commonly found in sera with myositis, and some are highly specific for this disorder. Each myositis-specific antibody defines a group of myositis patients with distinctive clinical features.

About 30 % of adults with myositis have antibodies to an aminoacyl transferase, and in at least 80% of cases the antibodies are directed to HRS. Anti-Jo-1 antibodies are almost exclusively found in patients with myositis. They occur in primary polymyositis with a prevalence of 33%, in primary dermatomyositis with 25% and in secondary myositis associated with other connective tissue diseases with 15% prevalence. The onset of the disease is often acute with prominent systemic features such as fever.

Myositis is often severe although cases without clinical muscle involvement are reported. Interstitial pneumonitis is a prominent clinical manifestation which is the next most common clinical feature after myositis in anti-Jo-1 positive patients, being present in 50-90 % compared to <10% of other patient with polymyositis or dermatomyositis.

Other myositis-specific antibodies have been detected (prevalence <5%): antibodies against threonyl- (anti-PL-7), alanyl- (anti-PL-12), isoleucyl- (anti-OJ) and glycyl-tRNA synthetase (anti-EJ) e.g.

### ***Principle of the test***

Serum samples diluted 1:101 are incubated in the microplates coated with the specific antigen. Patient's antibodies, if present in the specimen, bind to the antigen. The unbound fraction is washed off in the following step. Afterwards anti-human immunoglobulins conjugated to horseradish peroxidase (conjugate) are incubated and react with the antigen-antibody complex of the samples in the microplates. Unbound conjugate is washed off in the following step. Addition of TMB-substrate generates an enzymatic colorimetric (blue) reaction, which is stopped by diluted acid (color changes to yellow). The rate of color formation from the chromogen is a function of the amount of conjugate bound to the antigen-antibody complex and this is proportional to the initial concentration of the respective antibodies in the patient sample.



Product Ref.	7113US
Product Desc.	Jo-1
Manual Rev. No.	007: 2026-01-12

### 3 Kit Contents

---

**To be reconstituted:**

- 5x Sample Buffer 1 vial, 20 ml - 5x concentrated (capped white: yellow solution)  
Containing: Tris, NaCl, BSA, sodium azide (preservative)
- 50x Wash Buffer 1 vial, 20 ml - 50x concentrated (capped white: green solution)  
Containing: Tris, NaCl, Tween, sodium azide (preservative)

**Ready to use:**

- Negative Control 1 vial, 1.5 ml (capped green: yellow solution)  
Containing: Human serum (diluted), sodium azide (preservative)
- Positive Control 1 vial, 1.5 ml (capped red: yellow solution)  
Containing: Human serum (diluted), sodium azide (preservative)
- Cut-off control 1 vial, 1.5 ml (capped blue: yellow solution)  
Containing: Human serum (diluted), sodium azide (preservative)
- Calibrators 6 vials, 1.5 ml each 0, 3, 10, 30, 100, 300 U/ml  
(color increasing with concentration: yellow solutions)  
Containing: Human serum (diluted), sodium azide (preservative)
- Conjugate 1 vial, 15 ml IgG (capped blue: blue solution)  
Containing: Anti-human immunoglobulins conjugated to horseradish peroxidase
- TMB Substrate 1 vial, 15 ml (capped black)  
Containing: Stabilized TMB/H<sub>2</sub>O<sub>2</sub>
- Stop Solution 1 vial, 15 ml (capped white: colorless solution)  
Containing: 1M Hydrochloric Acid
- Microtiterplate 12x8 well strips with breakaway microwells  
Coating see paragraph 1

**Material required but not provided:**

Microtiter plate reader 450 nm reading filter and optional 620 nm reference filter (600-690 nm). Glass ware (cylinder 100-1000 ml), test tubes for dilutions. Vortex mixer, precision pipettes (10, 100, 200, 500, 1000 µl) or multipipette. Microplate washing device (multichannel pipette or automated system), adsorbent paper. Our tests are designed to be used with purified water according to the definition of the United States Pharmacopeia and the European Pharmacopeia.

### 4 Storage and Shelf Life

---

Store all reagents and the microplate at 2-8°C/35.6-46.4°F, in their original containers. Once prepared, reconstituted solutions are stable for 1 month at 2-8°C/35.6-46.4°F, at least.

**Reagents and the microplate shall be used within the expiry date indicated on each component, only. Avoid intense exposure of TMB solution to light. Store microplates in designated foil, including the desiccant, and seal tightly.**



Product Ref.	7113US
Product Desc.	Jo-1
Manual Rev. No.	007: 2026-01-12

## 5. Precautions of Use

---

### 5.1 Health hazard data

**THIS PRODUCT IS FOR IN VITRO DIAGNOSTIC USE ONLY.** Thus, only staff trained and specially advised in methods of in vitro diagnostics may perform the kit. Although this product is not considered particularly toxic or dangerous in conditions of normal use, refer to the following for maximum safety:

#### **Recommendations and precautions**

This kit contains potentially hazardous components. Though the kit reagents are not classified being irritant to eyes and skin we recommend to avoid contact with eyes and skin and wear disposable gloves.

Do not smoke, eat or drink when manipulating the kit.

Do not pipette by mouth.

All human source material used for some reagents of this kit (controls, standards e.g.) has been tested by approved methods and found negative for HbsAg, Hepatitis C and HIV 1. However, no test can guarantee the absence of viral agents in such material completely. Thus, handle kit controls, standards and patient samples as if capable of transmitting infectious diseases and according to national requirements.

### 5.2 General directions for use

Do not mix or substitute reagents or microplates from different lot numbers. This may lead to variations in the results.

Allow all components to reach room temperature (20-26°C/68-78.8°F) before use, mix well and follow the recommended incubation scheme for an optimum performance of the test.

Never expose components to higher temperature than 37°C/ 98.6 °F.

Always pipette substrate solution with brand new tips only. Protect this reagent from light. Never pipette conjugate with tips used with other reagents prior.

**A definite clinical diagnosis should not be based on the results of the performed test only, but should be made by the physician after all clinical and laboratory findings have been evaluated. The diagnosis is to be verified using different diagnostic and medicinal methods if the patient has got infectious diseases accompanied by medication.**

## 6 Sample Collection, Handling and Storage

---

Use preferentially freshly collected serum samples. Blood withdrawal must follow national requirements.

Do not use icteric, lipemic, hemolysed or bacterially contaminated samples. Sera with particles should be cleared by low speed centrifugation (<1000 x g). Blood samples should be collected in clean, dry and empty tubes. After separation, the serum samples should be used immediately, respectively stored tightly closed at 2-8°C/35.6-46.4°F up to three days, or frozen at -20°C/-4°F for longer periods.

## 7 Assay Procedure

### 7.1 Preparations prior to pipetting

Dilute concentrated reagents:

Dilute the concentrated sample buffer 1:5 with distilled water (e.g. 20 ml plus 80 ml).

Dilute the concentrated wash buffer 1:50 with distilled water (e.g. 20 ml plus 980 ml).

#### Samples

Dilute serum samples 1:101 with sample buffer (1x).

e.g. 1000 µl sample buffer (1x) + 10 µl serum. Mix well !

#### Washing

Prepare 20 ml of diluted wash buffer (1x) per 8 wells or 200 ml for 96 wells

(e.g. 4 ml concentrate plus 196 ml distilled water).

#### Automated washing:

Consider excess volumes required for setting up the instrument and dead volume of robot pipette.

#### Manual washing:

Discard liquid from wells by inverting the plate. Knock the microwell frame with wells downside vigorously on clean adsorbent paper. Pipette 300 µl of diluted wash buffer into each well, wait for 20 seconds. Repeat the whole procedure twice again.

#### Microplates

Calculate the number of wells required for the test. Remove unused wells from the frame, replace and store in the provided plastic bag, together with desiccant, seal tightly (2-8°C/35.6-46.4°F).

### 7.2 Work flow

- Pipette 100 µl of each patient's diluted serum into the designated microwells.
- Pipette 100 µl calibrators OR cut-off control and negative and positive controls into the designated wells.
- Incubate for 30 minutes at room temperature (20-26°C/68-78.8°F).
- Wash 3x with 300 µl washing buffer (diluted 1:50).
- Pipette 100 µl conjugate into each well.
- Incubate for 15 minutes at room temperature (20-26°C/68-78.8°F).
- Wash 3x with 300 µl washing buffer (diluted 1:50).
- Pipette 100 µl TMB substrate into each well.
- Incubate for 15 minutes at room temperature (20-26°C/68-78.8°F), in the dark.
- Pipette 100 µl stop solution into each well, using the same order as pipetting the substrate.
- Incubate 5 minutes minimum.
- Agitate plate carefully for 5 sec.
- Read absorbance at 450 nm (optionally 450/620 nm) within 30 minutes.



Product Ref.	7113US
Product Desc.	Jo-1
Manual Rev. No.	007: 2026-01-12

## 8 Quantitative and Qualitative Interpretation

For **quantitative interpretation** establish the standard curve by plotting the **optical density (OD) of each calibrator (y-axis)** with respect to the corresponding concentration values in **U/ml (x-axis)**. For best results we recommend log/lin coordinates and 4-Parameter Fit. From the OD of each sample, read the corresponding antibody concentrations expressed in **U/ml**.

<b>Normal Range</b>	<b>Positive Results</b>
<b>≤ 15 U/ml</b>	<b>&gt; 15 U/ml</b>

### Example of a standard curve

We recommend pipetting calibrators in parallel for each run.

<b>Calibrators IgG</b>	<b>OD 450/620 nm</b>	<b>CV %</b>
0 U/ml	0.051	0.0
3 U/ml	0.136	1.8
10 U/ml	0.334	2.2
30 U/ml	0.635	2.9
100 U/ml	1.278	2.4
300 U/ml	2.292	0.8

### Example of a calculation

<b>Patient</b>	<b>Replicate (OD)</b>	<b>Mean (OD)</b>	<b>Result (U/ml)</b>
P 01	0.840/0.849	0.845	48.4
P 02	0.351/0.376	0.364	13.6

For lot specific data, see enclosed quality control leaflet. Medical laboratories might perform an in-house Quality Control by using own controls and/or internal pooled sera, as foreseen by EU regulations. **Do not use this example for interpreting patients results!**

Each laboratory should establish its own normal range based upon its own techniques, controls, equipment and patient population according to their own established procedures.

For **qualitative interpretation** read the optical density of the cut-off control and the patient samples. Compare patient's OD with the OD of the cut-off control. All samples which are higher than cut-off are considered positive.

<b>Negative:</b>	<b>OD patient &lt; OD cut-off</b>
<b>Positive:</b>	<b>OD patient &gt; OD cut-off</b>



Product Ref.	7113US
Product Desc.	Jo-1
Manual Rev. No.	007: 2026-01-12

## 9 Technical Data

<b>Sample material:</b>	serum
<b>Sample volume:</b>	10 µl of sample diluted 1:101 with 1x sample buffer
<b>Total incubation time:</b>	60 minutes at room temperature (20-26°C/68-78.8°F)
<b>Calibration range:</b>	0-300 U/ml
<b>Analytical sensitivity:</b>	1.0 U/ml
<b>Storage:</b>	at 2-8°C/35.6-46.4°F use original vials, only
<b>Number of determinations:</b>	96 tests

## 10 Performance Data

### 10.1 Analytical Sensitivity

The analytical sensitivity of this kit has been found at 1.0 U/ml.

### 10.2 Specificity and sensitivity

The microplates are coated with **recombinant human histidyl-tRNA synthetase (Jo-1)**. No cross reactivities to other autoantigens have been found. The diagnostic sensitivity of Jo-1 antibodies is about 25%.

A study with 20 Anti-Jo1 positive and 51 negative sera (from patients with various rheumatic disorders) and the **AESKULISA® Jo-1** is shown in the table below.

clinical data for Jo-1	results from the AESKULISA Jo-1		
		positive	negative
	positive	20	0
negative	0	51	

100% agreement

### 10.3 Linearity

Chosen sera have been tested with this kit and found to dilute linearly. However, due to the heterogeneous nature of human autoantibodies there might be samples that do not follow this rule.

Sample No.	Dilution measured Factor	measured concentration (U/ml)	expected concentration (U/ml)	Recovery (%)
1	1 / 100	68.0	70.0	97.1
	1 / 200	33.3	35.0	95.1
	1 / 400	16.9	17.5	96.6
	1 / 800	8.3	8.8	94.3
2	1 / 100	226.0	230.0	98.3
	1 / 200	108.0	115.0	93.9
	1 / 400	54.0	57.5	93.7
	1 / 800	26.3	28.8	91.3

#### 10.4 Precision

To determine the precision of the assay, the variability (intra and inter-assay) was assessed by examining its reproducibility on three serum samples selected to represent a range over the standard curve.

Intra-Assay			Inter-Assay		
Sample No.	Mean (U/ml)	CV (%)	Sample No.	Mean (U/ml)	CV (%)
1	15.3	1.0	1	19.2	2.0
2	63.3	7.5	2	66.0	4.5
3	112.9	9.3	3	112.9	9.9

#### 10.5 Calibration




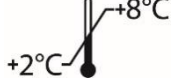

The **AESKULISA®**Jo-1 is calibrated against reference sera from the CDC Atlanta (Centers for Disease Control and Prevention). The results are expressed in U/ml.

## 11 Literature

---

- Nishikai and Reichlin (1980).**  
*Heterogeneity of precipitating antibodies in polymyositis and dermatomyositis.*  
 Characterization of the Jo-1 antibody system.  
 Arthritis Rheum 23: 881-888.
- Love LA, Leff RL, Fraser DD, Targoff IN, Dalakas M, Plotz PH, Miller FW (1991).**  
*A new approach to the classification of idiopathic inflammatory myopathy: myositis-specific autoantibodies define useful homogeneous patient groups.*  
 Medicine (Baltimore) 70: 360-374.
- Miller FW, Twitty SA, Biswas T, Plotz PH (1990a).**  
*Origin and regulation of a disease-specific autoantibody response. Antigenic epitopes, spectro type stability and isotype restriction of anti-Jo-1 autoantibodies.*  
 J Clin Invest 85: 468-475.
- Miller FW (1991).**  
*Humoral immunity and immunogenetics in the idiopathic inflammatory myopathies.*  
 Curr Opin Rheumatol 3: 902-019.
- Biswas T, Miller FW, Takagaki Y, Plotz PH (1987).**  
*An enzyme-linked immunosorbent assay for the detection and quantitation of anti-Jo-1 antibody in human serum.*  
 J Immunol Methods 98: 243-248.

## 12 Regulatory Symbols




<b>IVD</b>	For in vitro diagnostic use
<b>REF</b>	Catalog number
<b>LOT</b>	Lot
<b>UDI</b>	Unique Device Identifier
	96 tests
	See instructions for use
	Use by
	Store at 2-8°C (35.6-46.4°F)
	Manufactured by
<b>CON +</b>	Positive Control
<b>CON -</b>	Negative Control
<b>CAL</b>	Calibrator
<b>CO-CAL</b>	Cut off Calibrator
<b>CONJ</b>	Conjugate
<b>MP</b>	Coated microtiter plate
<b>WASHB 50x</b>	Wash buffer
<b>SUB</b>	Substrate buffer
<b>STOP</b>	Stop solution
<b>SB 5x</b>	Sample buffer
<b>Rx only</b>	For Prescription Use only

## Annex

### A: Pipetting Scheme

	Calibrators A-F	Controls	Samples
<b>Pipette</b>	Calibrators A-F	100 µl each	-
<b>Pipette</b>	Controls	-	100 µl each
<b>Pipette</b>	Prediluted samples (1:101)	-	100 µl each
Incubate	<b>30 min at room temperature (20-26°C/68-78.8°F)</b>		
Decant	<b>Wash 3x with 300 µl of wash buffer (1x)</b>		
<b>Pipette</b>	Conjugate	100 µl	100 µl
Incubate	<b>15 min at room temperature (20-26°C/68-78.8°F)</b>		
Decant	<b>Wash 3x with 300 µl of wash buffer (1x)</b>		
<b>Pipette</b>	Substrate	100 µl	100 µl
Incubate	<b>15 min at room temperature (20-26°C/68-78.8°F), in the dark.</b>		
<b>Pipette</b>	Stop Solution	100 µl	100 µl
Incubate	<b>5 min at room temperature (20-26°C/68-78.8°F)</b>		
<b>Agitate plate for 5 seconds and read OD at <math>\lambda</math>450nm (optionally <math>\lambda</math> 450/620 nm) within 30 minutes. Resulting color is stable for 30 minutes, at least.</b>			

**B: Test Procedure**

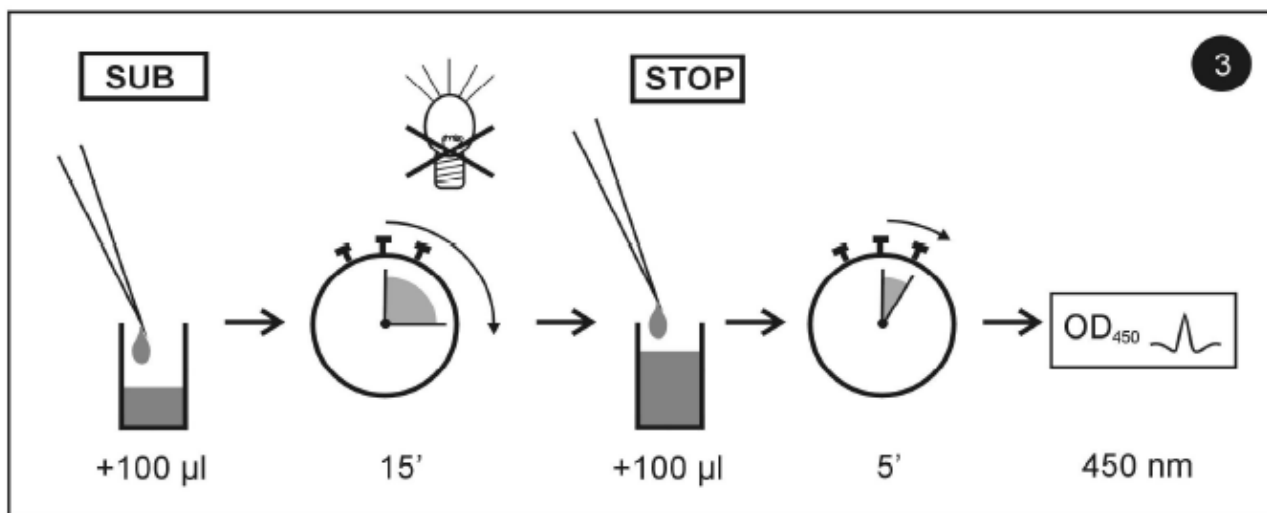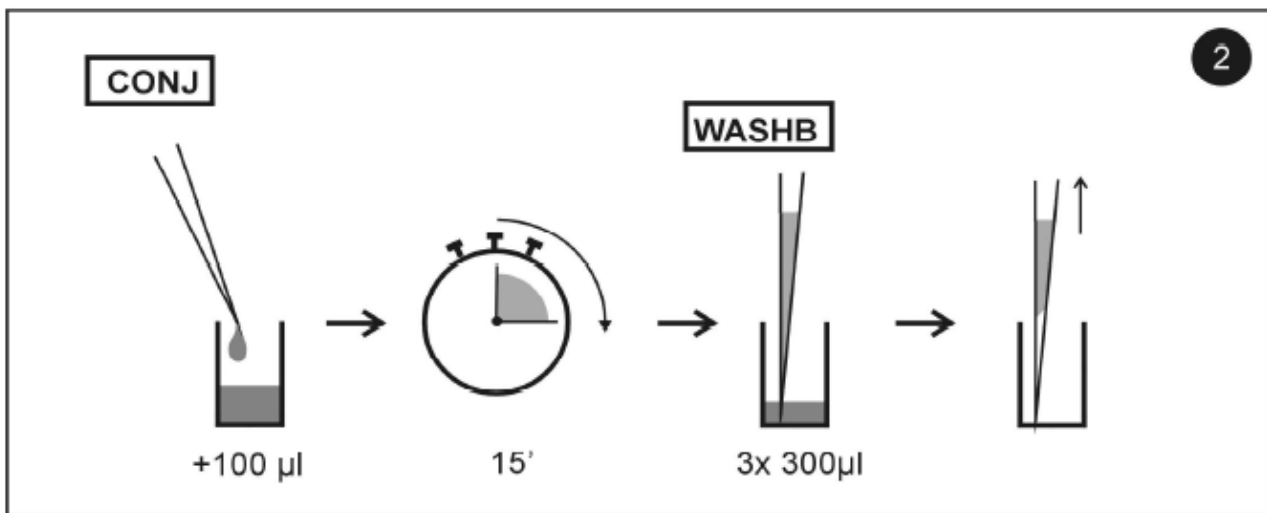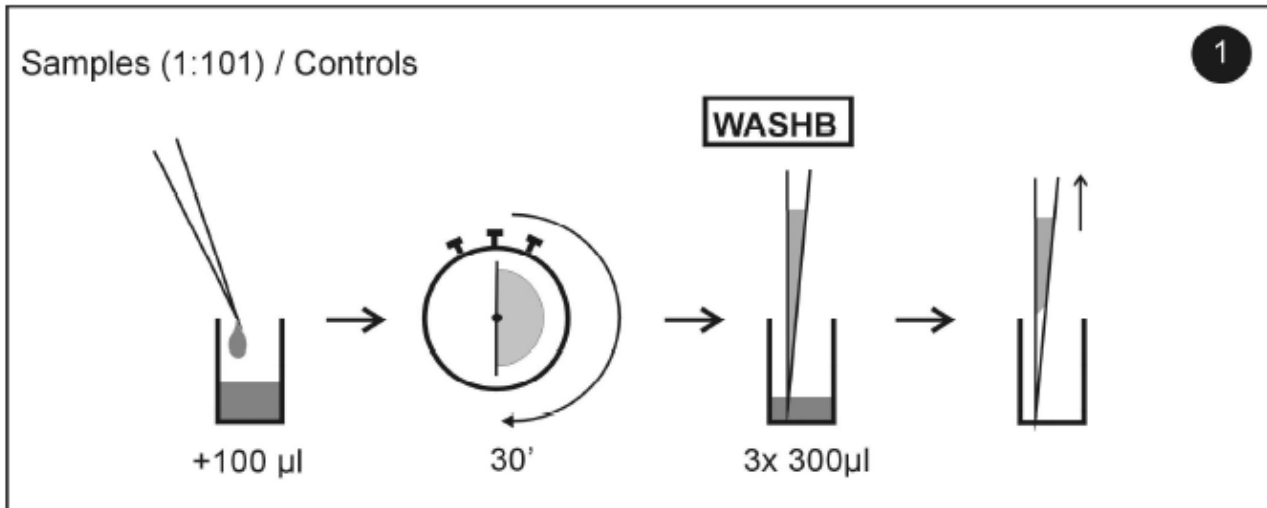



Product Ref.	7113US
Product Desc.	Jo-1
Manual Rev. No.	007: 2026-01-12

**C: Test Protocol**

Assay/Test: \_\_\_\_\_ Incubation/ Inkub. : 1. \_\_\_\_\_ min Date/Datum: \_\_\_\_\_  
 Temperature/Temperatur: \_\_\_\_\_ °F \_\_\_\_\_ °C Signature/Unterschrift: \_\_\_\_\_  
 Name: \_\_\_\_\_ 2. \_\_\_\_\_ min  
 3. \_\_\_\_\_ min

	1	2	3	4	5	6	7	8	9	10	11	12
A												
B												
C												
D												
E												
F												
G												
H												