


AESKU.RAPID SARS-CoV-2 Antigen Test


TEST PERFORMANCE

1

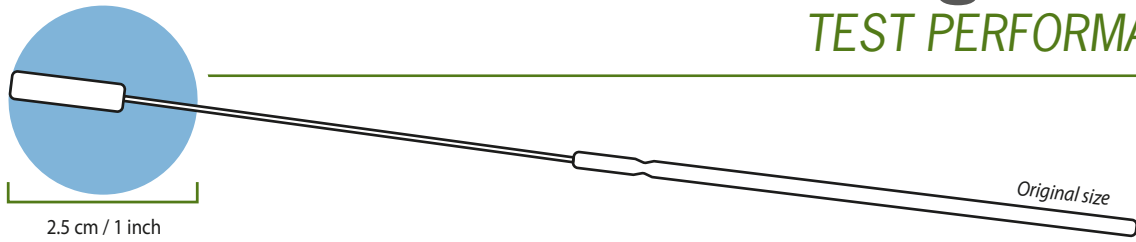

Carefully insert the swab **about 2.5 cm / 1 inch deep into one nostril**. Roll the swab **5 times** along the nasal mucosa. **Repeat** this process with the same swab in the **other nostril**.

2

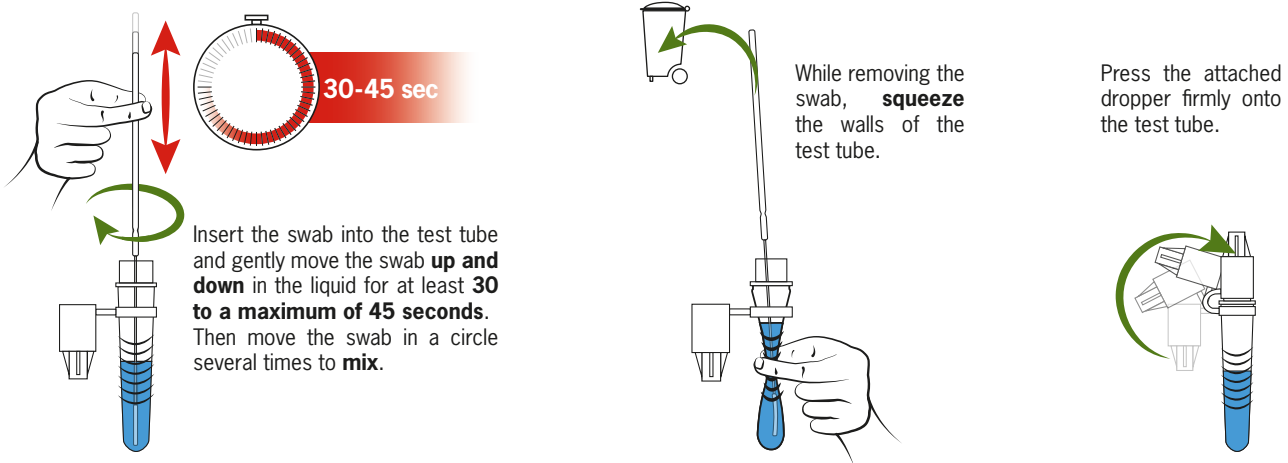


3



4

Assess the test result **within** a time window of **5 minutes**.

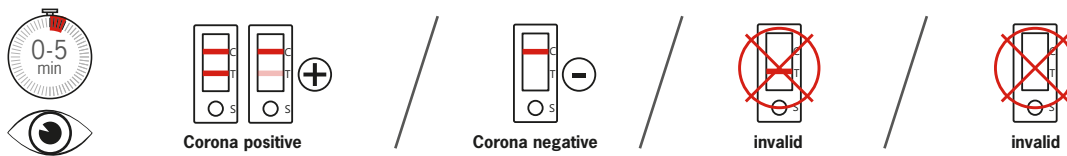

! **If your test result is invalid**
If you receive an invalid test result, please repeat the test with a new test kit!

! **If your test result is positive**
Positive results must be reported immediately to the appropriate health department following local guidelines. The positive test result must always be verified by a PCR test at an established physician – as soon as possible. Until the result of this test is known, you and your household should isolate yourself according to the guidelines.